




INSTRUCTIONS FOR USE

T-RING FLEX Application:

1) Slide onto digit.*

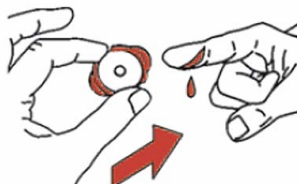

2) Bleeding stopped.
Perform procedure.



3) For large flaps and deformities stretch to enlarge opening until past wound, then slide to base of digit.



* Do not apply if risk of further damage of tissue.

* Application not to exceed 90 minutes.

* Do not use if package is damaged.


T-RING FLEX Removal Options:

1) Stretch T-RING FLEX
and remove.



OR

2) Cut to Remove.

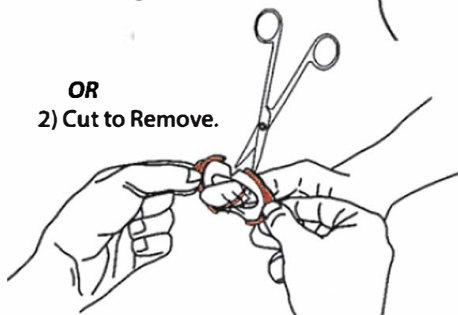

APPLICATION NOT TO EXCEED 90 MINUTES