


T-STRIP INSTRUCTIONS:


1. CONTROL

- Slide **T-RING** onto Finger
- Bleeding instantly stops
- One size fits all

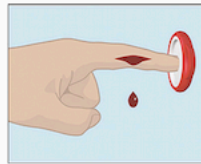


2. CLEAN

- Rinse Thoroughly
- Examine wound
- Dry with gauze

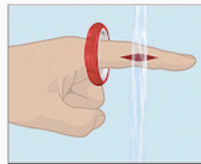


3. CLOSE

- Remove **T-Strip** backing
- Apply T-Strip to one side of wound
- Gently pull across wound to neatly approximate edges

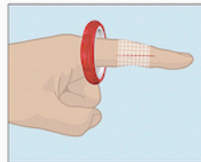

4. SEAL

- Use a Topical Skin Adhesive to completely coat T-Strip
- **REMOVE T-RING!** (60 min max)
- Apply non-adhesive bandage

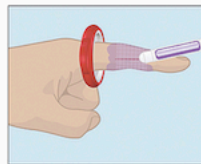

Learn more at: www.theTstrip.com

*For injuries not involving fingers, begin at Step #3